

IDEXX Laboratories UN 3373 Category B Packaging Instruction

Biological Substance, Category B (UN 3373) Shipments

IDEXX UN3373 Category B Packaging System is designed to meet the strict requirements of International Civil Aviation Organization (ICAO) and International Air Transport Association (IATA) regulations and packaging instructions PI 650 for the safe transportation of UN3373 Biological substances Category B classified substances.

Not adhering to these guidelines might result in delays or even rejecting shipments!

How to pack according to UN 3373?



1. Wrap samples into absorbent material
 - Multiple samples must be wrapped individually to prevent contact between them.
 - Please use sample tubes provided by IDEXX Laboratories.



2. Place wrapped samples and corresponding request form into one secondary packaging bag (95 kPa bag).
 - **NOTE: only one request form per bag!**
 - Place the request form inside the bag (avoid multiple folding).



3. Seal packaging



4. Place secondary packaging bag into the transport box.
 - Ensure the box contains sufficient cushioning material to prevent movement of the samples



5. Close the box properly
 - Please place a DHL Air-Waybill on the top of the IDEXX shipping box
 - Ensure the UN 3373 marking stays visible at any time!



Important for non EU countries: For countries outside the European Union a pro forma invoice has to be added to the shipment. You can download a standard version on www.idexx.eu

Contacts and Resources (Your quick access to DHL online-booking: <http://bit.ly/1cvC8c1>)

To find the phone number of your local DHL customer service in your country, please use <http://bit.ly/1abDxWn> and select your country.

